Visio-AMTRIX®: Instruction for use

Name of the product: Visio-AMTRIX®: Human amnion, devitalized.

Composition: Allogenic tissue from human fetal membrane.

Description: Dried grafts of different sizes

Reference	Characteristics	Sizing
AD50	Collagen layer	Diam 5 cm
AD20	Collagen layer	Diam 2 cm
AA33	Collagen layer	3 cm x 3 cm
AA44	Collagen layer	4 cm x 4 cm
AA64	Collagen layer	6 cm x 4 cm

Therapeutic indications: Rebuilding of the corneal surface or surface of the conjunctiva of the eye. Further applications in other operative disciplines are also conceivable.

Contraindications: Grafting in necrotic operation sites is contra-indicated.

Method of administration:

Visio-AMTRIX is sterile when supplied and should be considered as such unless the primary packaging has been opened.

The graft is ready to use and may be applied to the eye without being rehydrated.

To identify the different sides of the amnion: the epithelial layer is located on the underside.

Briefly, a speculum is positioned to retract the eyelids. The dehydrated sheet of amniotic membrane is placed and smoothed over the corneal surface. Then a large-diameter contact lens is situated atop the membrane to secure it in place.

Special warnings and precautions for use:

Visio-AMTRIX must be used only by trained and qualified personnel, who has carefully read the instructions for use. Aseptic conditions must be used when opening the protective packaging; the product shall be used immediately.

The product is sterile when delivered. It has been sterilized in the final packaging in accordance with current standards using a dose of 25kGy of gamma radiation.

Before using the product, it is essential to check the integrity of the packaging and the expiry date, marked on the label.

Do not re-sterilize any part of the components.

Interaction with other medicinal products and other forms of interaction:

No interaction studies have been performed.

Fertility, pregnancy and lactation:

No effects during pregnancy and lactation are anticipated, due to the nature of the product. No risks for pregnant women and nursing mothers have been identified due to Visio-AMTRIX.

Effects on ability to drive and use machines:

Visio-AMTRIX itself has no influence on the ability to drive or the ability to use machines.

Undesirable effects: Side effects commonly associated with a surgical procedure can not always be delineated from those related to the graft itself. By using Visio-AMTRIX, loss of visual acuity or ulcer recurrence could appear. These side effects may sometimes be pathological.

Shelf life:

Visio-AMTRIX grafts are stable at room temperature between +15 to +35°C for 5 years.

Special precautions for storage:

The product must be stored in its original packaging and the storage conditions must ensure, that the packaging and its components are not compressed.

It must be stored away from dust and light in a dry place with an ambient temperature (+15°C to + 35°C).

Do not drop or crush the boxes. The primary and secondary packaging of Visio-AMTRIX consists of a peel pouches.

One side is high-density polyethylene (Tyvek®) and the other one is polyethylene terephthalate / polyethylene (PET/PE). The external packaging is made of an over pack box for protection against mechanical shock.

Special precautions for disposal:

Visio-AMTRIX product is sterile when supplied and should be considered as such unless the primary packaging has been opened.

Shelf life: 5 years in the original and unopened packaging.

Special precautions for storage: +15°C to +35 °C.

VIGILANCE: Any adverse event or incident should be reported without delay.

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